

Pandoria Plus

Important information on upholstery covering care:

Cleaning:

Cleaning is understood as the removal of visible dirt and invisible organic material to prevent microorganisms from multiplying.

Use a warm, mild soapy water solution and a microfibre cloth or a soft hand brush for cleaning. We recommend that you regularly clean the surface of your upholstery.

Please do not use solvents, chlorides, polishing agents, chemical cleaning agents or wax polishes!
A special cleaning agent can be obtained from Dewert.

Our recommendation:



Cleaning kit for Skai products for careful cleaning of upholstery (Art. no. 050-1000)

Consists of:

- Spray bottle with 200 ml special Skai cleansing agent
- 1 microfibre care cloth 40 x 40 cm

Disinfection:

Disinfection is understood as the killing or inactivation of pathogens to prevent infections in humans.

Do not use any products containing solvents for this purpose either!

Disproportionate use of disinfectants affects the life of the upholstery covering.

The following disinfectants (see rear) have been tested for compatibility and can be used in the dosage indicated (subject to any changes in the composition of the disinfectants).

Please note:

It is essential to ensure proper handling.

Non-observance can lead to brittleness, discolouration and cracking of the upholstery covering within a short space of time! In this case, the warranty will be null and void.

Staining caused by jeans or other textiles is also excluded from the warranty!

Disinfectants for Pandoria Plus

Designation	RKI list	VAH list	Manufacturer
Incidin Plus	x	x	Ecolab GmbH
Incidin Pro		x	Ecolab GmbH
Incidin Aktive	x	x	Ecolab GmbH
Incidin OxyWipes S		x	Ecolab GmbH
Microbac forte		x	Bode Chemie
Kohrsolin FF		x	Bode Chemie
Bacillol 30 foam		x	Bode Chemie
Dismozon plus	x	x	Bode Chemie
CosiMed Konzentrat			CosiMed GmbH
Cleanisept/Wipes		x	Dr. Schumacher GmbH
Optisept	x	x	Dr. Schumacher GmbH
Biguanid Fläche N		x	Dr. Schumacher GmbH
Descosept PUR Wipes RTU		x	Dr. Schumacher GmbH
Descosept Sensitive Wipes		x	Dr. Schumacher GmbH
Descosept Spezial Wipes		x	Dr. Schumacher GmbH
Ultrasol active		x	Dr. Schumacher GmbH
Meliseptol foam pure		x	B.Braun Melsungen
Meliseptol rapid		x	B.Braun Melsungen
FD 366		x	Dürr Dental AG
FD 300		x	Dürr Dental AG
Aldasan 2000	x	x	Lysoform, Dr. Rosemann GmbH
Lysoformin spezial	x	x	Lysoform, Dr. Rosemann GmbH
Franco-Cid N		x	Franken Chemie GmbH & Co.KG
ASCEA-des			Aquagenius Schweiz GmbH
Mikrozyd sensitive liquid		x	Schülke & Mayr
Perform	x	x	Schülke & Mayr
Mikrozyd universal liquid		x	Schülke & Mayr
Terralin protect	x	x	Schülke & Mayr
Apesin rapid		x	Tana Chemie GmbH
Biguacid S		x	Antiseptica, Dr. H.-J. Molitor GmbH
Descogen Liquid r.f.u.		x	Antiseptica, Dr. H.-J. Molitor GmbH

All disinfectants listed are effective against SARS-CoV-2 according to the manufacturer's information.

Attention:

Please always observe the relevant manufacturer's dosage instructions when using disinfectants!